


# How to Take Oral Temperatures

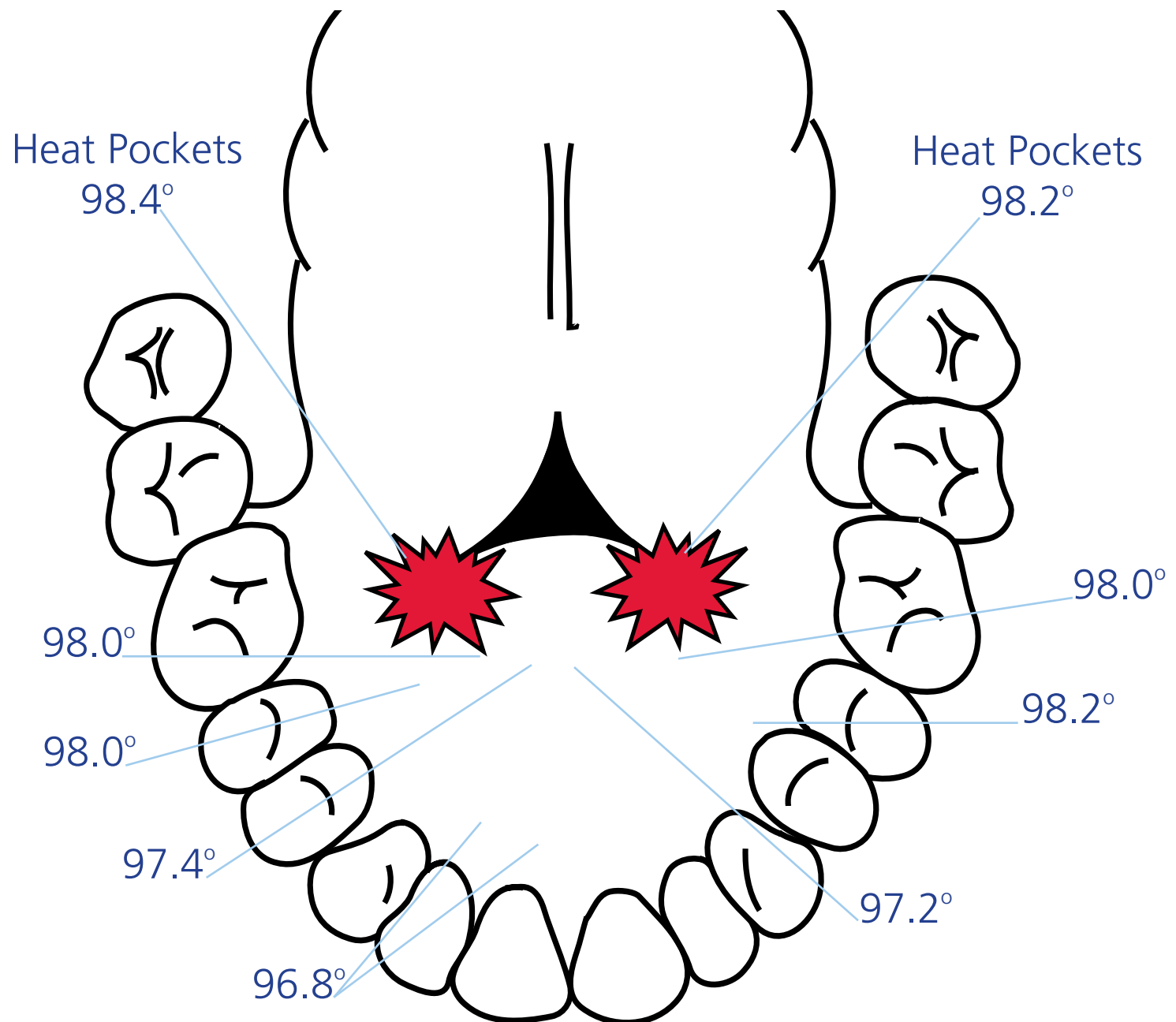

**Accurate temperatures can only be obtained in the “Heat Pocket”!**

## Temperature Variables: Floor of the Mouth

1. Ensure the blue isolation chamber and probe are properly attached
2. Verify Oral Mode is indicated on the LCD screen
3. Apply new Filac™ 3000 probe cover
4. Insert probe tip deep into the sublingual pocket next to the frenulum liguae - the “Heat Pocket”
5. Hold the probe steady with the patient’s mouth closed until a long beep sounds, indicating the temperature measurement is complete and final reading is displayed
6. Eject probe cover into a medical waste container and return probe to the isolation chamber

## Normal Temperature Ranges

Temperature Site	0-2 Years	3-10 Years	11-65 Years	>65 Years
Oral		95.9°- 99.5°F 35.3 - 37.5°C	97.6°- 99.6°F 36.7 - 37.8°C	96.4°- 98.5°F 35.7 - 36.9°C

For more information call 800.962.9888  
or visit [www.covidien.com](http://www.covidien.com)