

Kangaroo ePump™ Enteral Feeding Pump

Set Loading Instructions for Use

Follow proper procedures for filling or spiking feed container. Press power button in lower right-hand corner to turn pump on.



Grasp thumb tab on valve and insert ring retainer directly into left pocket. Press down on thumb tab to ensure valve is fully seated. Thumb tab must align with white line on left.



Grasp black ring retainer, gently wrap tubing around rotor, insert retainer directly into right pocket. **DO NOT OVERSTRETCH TUBING.**



Once the feeding set is loaded, close the blue door. The pump is now ready for normal operation.

Kangaroo ePump™ Enteral Feeding Pump

Hang Height Instructions for Use



In order to achieve proper accuracy the bottom of the feeding set container must be 18 inches above the top of the feeding pump.

